prosetti
Medical Equipment Solutions

RESCUE Life

Think different, **be** different

DEFIBRILLATOR 

TECHNOLOGY FOR DIAGNOSIS AND EMERGENCY

DESIGN

Born from expert Italian designers, its strength is simplicity and beauty. Attractive line, ergonomic form, studied in every detail, Rescue Life is the first professional defibrillator totally MADE IN ITALY that respects the traditions of elegance and comfort. Thanks to the lightweight and transport facility this defibrillator is the perfect device in the hard challenges. Quality and safety is the mix of our expertise, applied to Rescue Life as well to all our products line. The IPX4 standard permits the operation under adverse climatic situations.

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TECHNOLOGY

Rescue Life is a new top product evolved from the improvements to the defibrillators manufactured by Progetti. From more than 25 years' experience, Rescue Life is the advanced solution for emergency medicine. Ergonomic form, extremely robust, it is the first professional defibrillator totally Made in Italy. Auto-test mode allows to verify the proper functions of the device, battery status and charge-discharge system with the possibility of real time data management and report printing.

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USABILITY

Close to the needs of the modern emergency medicine, Rescue Life guarantees impact-proof larger TFT LCD color 7" display, powerful battery up to 5 hours continuous monitoring and different product configurations. Individuals can choose from basic to fully equipped models among the available optional: up to 12 leads ECG monitoring, AED & Advisory working mode, external pacemaker, pulse oximeter and non-invasive blood pressure.

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A DIFFERENT POINT OF VIEW

The combined defibrillator monitor Rescue Life is designed to be used by out-of hospital and hospital users. Having an innovative and fully featured design, it provides for defibrillation and monitoring in a single portable device.

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DISPLAY

Brilliant, high-resolution TFT LCD color 7" 800x480 dots display allows the simultaneous visualization in real time of 3, 6 or 12 leads ECG and plethysmograph waveform with SpO2 sensor connected.

ECG leads error message when cable is disconnected.

Customizable ECG display traces in setup menu.

Three operating modes: manual, AED, Advisory. In Advisory mode Rescue Life alerts when the shock is necessary using voice prompts and display messages, leaving the energy level and charging choice to the operator.

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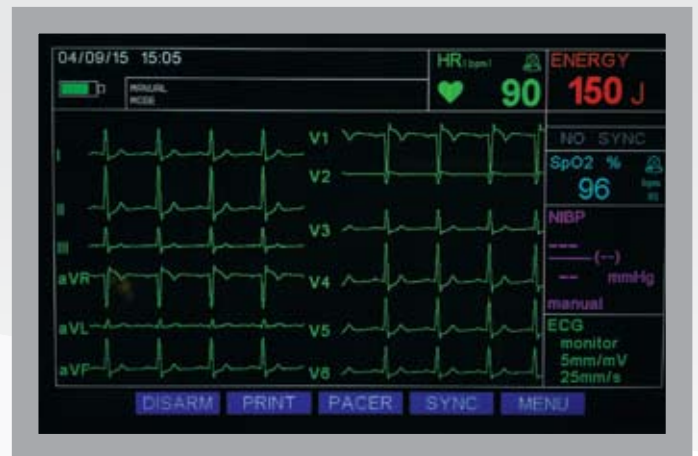
FAST CONNECTION

Users can choose between ergonomic reusable paddles, suitable for the manual defibrillation in both adult and pediatric patients or disposable pads for AED defibrillation. Both types are equipped with a single fast lock connection for reliable rescue operations. The ECG can be detected via 3, 5 or 10 leads ECG cable and the optional SpO2 sensor is available for detecting blood oxygen saturation. Rescue Life can be equipped with internal reusable defibrillation paddles with interchangeable different sizes.

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SENSIVITY IN ECG DETECTION

Rescue Life can filter the ECG signal automatically thanks to two modes:

DIAGNOSTIC: ECG signal processing recommended for interpretation and diagnosis of ECG traces by the qualified personnel.

MONITORING: digital filtered ECG signal, rejects the most parts of noises and artefacts caused by patient motions or by other physiological process of the body. These two modes may facilitate the ECG detection and help the qualified personnel to take important decisions for the diagnosis of heart diseases.

OPTIONALS

Optimized and comfortable carrying bag, SpO2 module, multifunction disposable defibrillation pads for adult and pediatric, 10 leads ECG cable, pacemaker module, NIBP module, mounting bracket 10G tested.

MEMORY AND DATA MANAGEMENT

Simple commands, extremely intuitive and easy to use. Immediate access to main setup parameters using only the trim knob and function keys. The internal memory is able to record more than 300 hours of ECG data and events. The data are recallable and printable on the integrated thermal printer. Printing can be done in manual or in automatic mode. In the automatic mode the printing initiates when the energy charging starts (recording the ECG signal before the shock) and will continue 10 seconds after the shock was delivered.

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TECHNICAL SPECIFICATIONS

GENERAL REQUIREMENTS

Dimension	• 369 x 240 x 340 mm
Weight	• 5,5 Kg with battery
Electrical	• AC 100-240 VAC, 50/60 Hz - inverter 12V car adapter (optional) • Rechargeable 15.0 V - 3.2 Ah Lithium-ion battery
Battery Operation Time	• > 5 hours
Battery Charge Time	• Max. 3 hours
Capacity	• 150 shocks at 230J with battery fully charged
Software	• Voice and texts multi language
Memory	• Internal flash 4 Gb up to 300 hrs of ECG data, events and usage review
Review and printing database	• Review and printing of archived events

ENVIRONMENTAL

Temperature	• Operational -5° ~ 46° C • Storage -20° ~ 50° C • Relative humidity 10 ~ 95%
Isolation	• ECG type CF, Defibrillation type BF
Water proof	• Class IPX4

DEFIBRILLATION

Waveform	• Biphasic Truncated Exponential (BTE) • HiCAP technology (Large storage capacitor)
Impedance	• Compensated, range 25 - 200 Ohm
Charge time	• 6 sec for 230 J with battery fully charged

AED MODE

Energy	• Fixed energy at 200 J
Protocol	• ERC 2015 CPR Guidelines with voice and text prompts
Shockable rhythms	• VF with amplitude > 0.15 mV and VT with rhythm > 150 bpm
Sensitivity	• Shockable rhythm-VF > 95% • Shockable rhythm-VT > 75%
Specificity	• According to ERC 2010 • Normal sinusoidal rhythm > 99% • Asystole and other non-shockable rhythms > 95%

MANUAL MODE

Energy range	• 1 - 230 J (from 1-10 J in 1 J steps; from 10 to 230 J in 10 J steps)
Commands	• Multifunction trim knob. Charge and shock button directly in the front panel for hands free defibrillation
Paddles	• Reusable adult & pediatric defi paddles with charge/shock command • Disposable defi pads and internal defi paddles (optional)
Operating mode	• ECG « R » wave synchro or async mode
Indicators	• Battery and main led indicators • Clear and visible backlight color buttons

DISPLAY

Type	• High contrast TFT color LCD 7" (800X480 dots)
Information	• Operational information, ECG traces, SpO2 value, NIBP values, parameters, Heart Rate indicator, filters, alarms and battery status
Display mode	• Can select 3, 6, 12 ECG leads

ECG MONITORING

Bandwidth	• 0.5 - 120 Hz (-3 db) with filters off
CMRR	• > 95 db
Input impedance	• > 20 Mohm
Input	• Patient cable 5 leads (I, II, III, aVL, aVF, aVR, V) - standard • Patient cable 10 leads (I, II, III, aVL, aVF, aVR, V1.....V6) - optional
Gain	• 2,5, 5, 10, 20, 40 mm/mV with patient cable. AUTO with pads
Speed	• 5, 10, 25, 50 mm/sec
Filters	• 550/60 Hz, EMG Filter, base line
Monitoring Mode: bandwidth	• 0.6 - 40 Hz (-3 db)
Diagnostic Mode: bandwidth	• 0.05 - 120 Hz (-3 db)
HR range	• 20 - 300 bpm (~ 2%), digital readout on the display
Alarm	• HR max settable 250 bpm; HR min 20 bpm

SPO2 (OPTIONAL)

SpO2 range	• 0 - 100%
HR range	• 30 - 250 bpm
Accuracy	• 70 - 100% ~ 2% for adults with finger clip sensor
Alarm	• Adjustable min 50%

NIBP (OPTIONAL)

Technique	• Oscillometric
NIBP Accuracy	• Meets ANSI/AAMI SP10-2002, EN 1060-4
Patient Application	• Adult/Paediatric/Neonatal
Systolic Range mmHG, Neonatal	• Adult: 40-260 mmHG, Paediatric: 40-160 mmHG, Neonatal: 40-130 mmHG
Range MAP mmHG, Neonatal	• Adult: 26-220 mmHG, Paediatric: 26-133 mmHG, Neonatal: 26-110 mmHG
Diastolic Range mmHG, Neonatal	• Adult: 20-200 mmHG, Paediatric: 20-120 mmHG, Neonatal: 20-100 mmHG
PR RANGE	• 30-220 BPM
PR Accuracy	• +/-2% or +/-3 bpm
Operating Modes	• Manual, Long Term Automatic, Stat, Service
Auto Interval Periods	• 1,2,3,4,5,10,15,30,60 and 90 minutes

PRINTER

Type	• Integrated thermal 3 channels printer for ECG traces and events documentation hardcopy including HR/SpO2/NIBP values
Paper speed	• 5, 10, 25, 50 mm/sec
Paper width	• 58 mm
Operating model	• Manual, automatic (10" pre and post shock recording)

EXTERNAL PACEMAKER (OPTIONAL)

Type	• Rectangular wave
Operating mode	• Fixed on demand
Pulse rate	• 30 ppm to 250 ppm, adjustable in steps of 5 ppm
Impulse duration	• 22.5 ms
Pulse current	• 0 to 150 mA adjustable in steps of 5mA
Amplitude	• Max 150 V

STANDARDS & SAFETY

Standard	• IEC 60601-2-4; IEC 60601-1; IEC 60601-1-2
CE Mark EEC 93/42	• Medical device, Class IIb

Specifications and contents of this document subject to change without notice.

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We have been developing products for the prevention and treatment of heart diseases for more than 25 years. Thanks to an international network and to our presence in more than 60 countries all over the world, we commit ourselves to providing the best Italian technology in the medical field.

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Looking into the future with enthusiasm
is the way to create innovative instruments.
Being the ideal partner for those who want to make
a better world with their work is our purpose.

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PROGETTIMEDICAL.COM

progetti
Medical Equipment Solutions

PROGETTI S.r.l.
Strada del Rondello, 5
10028 Trofarello (Torino) - Italy
Ph. +39 011 644 738
Fax +39 011 645 822
info@progettimedical.com
www.progettimedical.com

CE
0068

ISO 9001
BUREAU VERITAS
Certification

Quality system ISO certified

Distributed by: